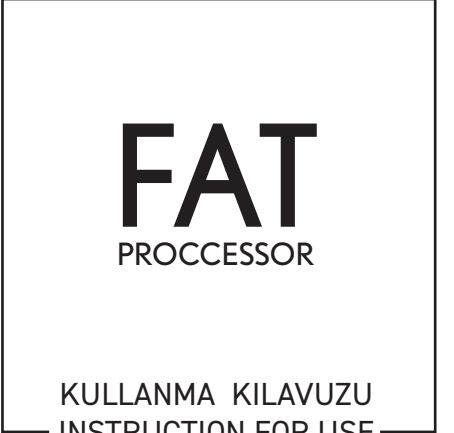
KULLANMA KILAVUZU

İÇİNDEKİLER

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INSTRUCTION FOR USE

TABLE OF CONTENTS

- The Name of The Medical Device
- The Name and The Address of The Manufacturer
- Product Content List
- Intended Use of The Product
- Indication for Use
- Possible Contraindicative Situations
- The Performance of The Device Intended by The Manufacturer
- Clinical Safety Information
- Patient Profile
- Residual Risk and Side Effect Information

EN

- Possible Adverse Events
- User Profile
- Medical Devices To Be Used With The Device
- Procedure
- Storage, Shelf Life and Handling Information
- Precautions, Warnings, Measures to Be Taken and Limitations
- Symbols and Explanations

THE NAME OR TRADE NAME OF THE MEDICAL DEVICE
FAT PROCESSOR

Fat Processor 2400, Fat Processor 1200, Fat Processor 600, Fat Processor 300, Fat Processor 200, Fat Processor 150, Fat Processor 130, Fat Processor 120, Fat Processor 100, Fat Processor 75, Fat Processor 50 and Fat Processor 41 are sterile single use devices.

THE NAME AND THE ADDRESS OF THE MANUFACTURER

T-Biyoteknoloji Laboratuvar Estetik Medikal Kozmetik San. ve Tic. Ltd. Şti
Dumlupınar Mah. Kuleli Cad. No:3 Nilüfer-Bursa / TÜRKİYE
Manufacturer ID No (SRN): TR-MF-000016403 **Phone:** +90 224 246 85 22
E-Mail: info@tllab.com.tr

PRODUCT CONTENT LIST

The content of the packaging is one of the following:

Product	Model	Ref	Color
Fat Processor 2400	FP2400	3101	PINK
Fat Processor 1200	FP1200	3102	GREEN
Fat Processor 600	FP600	3103	YELLOW
Fat Processor 300	FP300	3112	BL

Product	Model	Ref	Color
Fat Processor 200	FP200	3111	BLUE
Fat Processor 150	FP150	3110	RED
Fat Processor 130	FP130	3109	BLUE
Fat Processor 120	FP120	3108	BLUE
Fat Processor 100	FP100	3107	BLUE
Fat Processor 75	FP75	3106	BLUE
Fat Processor 50	FP50	3105	BLUE
Fat Processor 41	FP41	3104	BLUE

All products are sterile single use device.

INTENDED USE OF THE PRODUCT

FAT PROCESSOR is designed to be used for the safe and rapid emulsification and homogenizing of harvested adipose tissue into desired consistency or viscosity as deemed necessary by the clinical use requirements of fat graft.

INDICATION FOR USE

Indications for the use of FAT PROCESSOR include intradermal injection for the correction of fine wrinkles, intra and subdermal injection for the correction of sun damage and intra and subdermal injection for the correction of skin discoloration. The device can be used for a superficial fat replacement to restore thinning skin to a more youthful condition and to enhance skin quality.

POSSIBLE CONTRAINDICATIVE SITUATIONS

Do not use the device, if it does not meet the requirement(s) for use in the intended surgical procedure(s).

PERFORMANCE OF THE DEVICE INTENDED BY THE MANUFACTURER

FAT PROCESSOR is a closed filtration system used for the emulsification of fat tissue. The device includes two components either a stainless-steel blade-filter or a polyamide filter and a single-use polycarbonate processor. The processor has two female-to-female luer-lok to connect to two syringes of same size for the transfer of tissue. User shall manually transfer the tissue back and forth between syringes. User shall use the blade-filter in order from largest to smallest size until desired consistency and viscosity is achieved.

Adipose tissue has many natural reparative characteristics that help to support a healing environment and to rejuvenate the tissues throughout the body. Adipose tissue provides cushioning and support of normal tissues in the body by the action of growth factors up-regulating collagen and elastin production.

CLINICAL SAFETY INFORMATION

The safety and efficacy of this device have been proven by the Risk Analysis Report and Clinical Evaluation Report.

PATIENT PROFILE

Fat grafting is an autologous application therefore patient applications are inherently safe from infectious diseases such as HIV, Hepatitis, Creutzfeldt-Jakob disease. This application can be used safely and effectively on all patients. The safety and effectiveness of fat treatment have not been established in children and pregnant women.

RESIDUAL RISK AND SIDE EFFECTS INFORMATION

According to the risk assessment, risk management, risk report procedures of this device, residual risks are at an acceptable level, therefore the benefits gained from using the device are greater than its risks. With respect to the requirements of Essential Requirements, the risk management report showed that there were no

serious side effects and the performance of the product was verified by the clinical evaluation report.

POSSIBLE ADVERSE EVENTS

- Bruising and swelling
- Fat Necrosis
- Embolization due to injection of fat into veins or muscles
- Temporary or permanent nerve damage that may result in pain or numbness.
- Early or late postoperative infection.

USER PROFILE

This product should be used by qualified surgeons or surgeon-directed allied health care professionals with adequate training in the use of the device.

MEDICAL DEVICES TO BE USED WITH THE DEVICE

The user shall choose the syringe volume to be used based on the application and volume of fat to emulsify. FAT PROCESSOR is compatible with luer-lock syringes sized 1ml, 5ml, 10ml, 20ml, 50ml & 60ml. T-Biyoteknoloji recommends 10ml and 20ml syringes to be used with the processor for the ease of back and forward movements and avoiding loss of tissue in the processor. The user shall use two of the same size syringes for the process.

PROCEDURE

Fat Processor 2400, Fat Processor 1200, Fat Processor 600, Fat Processor 300, Fat Processor 200, Fat Processor 150, Fat Processor 130, Fat Processor 120, Fat Processor 100, Fat Processor 75, Fat Processor 50 and Fat Processor 41 are ready-to-use devices with the blade-filters embedded in.

- Open the box.
- Open the sterile blister packaging.

- Attach the desired size syringe holding the fat graft to the processor via luer connectors in the direction indicated by the arrow. Attach an empty syringe of the same size on the other luer connector.
- Manually force the fat graft back and forth between syringes until desired consistency and viscosity is reached.
- At the end of the process, ensure that the fat graft remains in the syringe that is on the outlet direction of the arrow marking on the processor.
- Use blade-filter 2400, 1200, 600, 300, 200, 150, 130, 120, 100, 75, 50 and 41 in order to further emulsify the fat graft.
- Transfer the fat graft into new syringe for further processes or injection.

SHELF LIFE

Product shelf life is 5 years.

STORAGE, PRODUCT LIFE AND HANDLING INFORMATION

- FAT PROCESSOR has been sterilized with Ethylene Oxide.
- Do not expose to direct sunlight.
- Store in a cool, dry place.
- Do not use if the inner sterile blister package is damaged or opened.
- Use by Date is indicated on the product label.
- Store at temperatures between 5°C and 30°C.

PRECAUTIONS, WARNINGS, MEASURES TO BE TAKEN AND LIMITATIONS

- FAT PROCESSOR is ready-to-use sterile devices, do not reuse them.
- Do not use the sterile components if the package is opened or damaged.
- Patient selection, patient safety precautions and appropriate anesthesia methods are required prior to the use of the device(s).
- The user should be thoroughly familiar with the equipment and the surgical procedure prior to using this device.

- The patient should be made aware of the general risks associated with treatment and possible adverse effects.
- Do not dispose of this product in the unsorted municipal waste stream.

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